


BRAIN GAUGE

QUICK START GUIDE





- 1** Download and Install the Brain Gauge App.
Visit www.corticalmetrics.com and click on the Download App button in the upper right-hand corner of the page. Follow the on-screen instructions to install and launch the application.

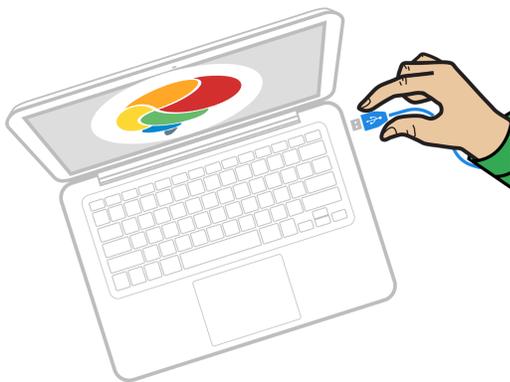

- 2** Plug in your Brain Gauge.
Connect the device to your computer's USB port. In some cases, you may need to allow time for the computer to recognize and download the driver for the device.

- 3** Create a new account.
Enter your name, email address, and a new password to set up an account. Store your log-in information in a safe place.

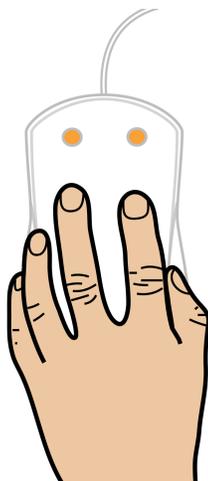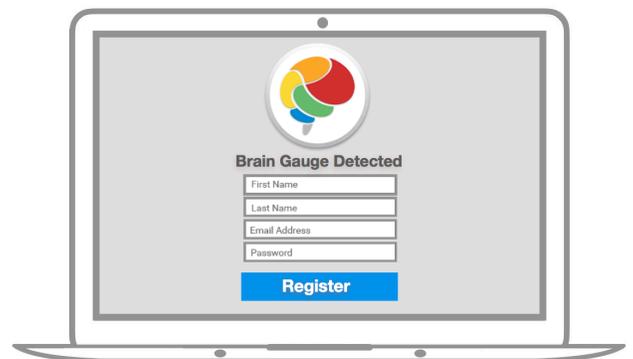

- 4** Place your hand on the device and begin testing.
Select or create a user and choose your desired tests. Rest your index and middle fingers on the sensors. Follow the on-screen instructions. Your results will be available after you complete the test.